

Informational Flyer

Reusable Community Masks

Level
3

Level 3: Fabric masks are for use while at work or in authorized excursions (during confinement), namely while indoors in the presence of multiple people.

Important Note:






Level 3 fabric masks are not medical devices or personal protective equipment.

Manufacturer: Emperitexir, Lda

Reference: MS3B2C

Composition: 100% cotton (2 layer ref: 3684/2)

Size: Adult

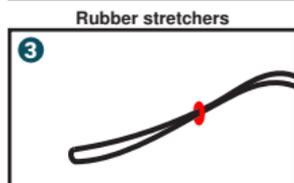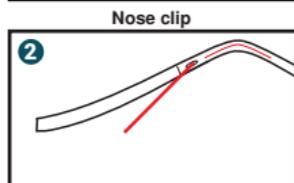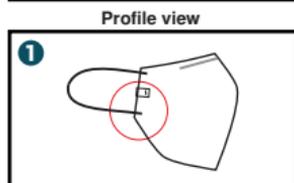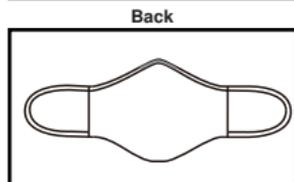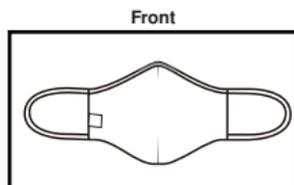

- 1 The tag (circled in the image) indicates the outside of the mask. Use the mask with the tag facing out.

Instructions for use:

- 2 Before each wash, remove the nose clip.

The mask must be washed before each new use.

The protection provided by the mask can only be guaranteed if the cleaning and preservation instructions are properly followed.

When not in use, keep the mask in a dry environment and protected from potential contamination.

The mask must be washed before its first use.

If the mask is defective or damaged, do not use.

At the end of the mask's lifetime, wash the mask as instructed in "Washing instructions" and discard it as non-dangerous fabric waste. If you do not wash the mask, discard it as biological waste.

- 3 Place the rubber stretchers to adjust the measurement of the elastics.

Remove the rubber stretchers before each wash.

Limitations for use:

This is not a medical device.

The mask must only be used when completely dry after washing.

The mask can only be used by a different person after being washed.

If the mask gets accidentally wet, please exchange it for a dry one

as soon as possible. The wet mask must be washed before reuse.

At the end of the recommended daily use, or as soon as you arrive home, the mask must be washed before reuse.




Example of donning a mask:



- A) Wash hands with soap and water or with hand sanitizer before donning a mask.
B) Don the mask with the internal side towards your face and the external side facing out.
C) Adjust the nose clip to your nose and make sure to cover your mouth, nose and chin, making sure there are no spaces between the mask and your face.
D) Do not touch the mask while wearing it. If you do, wash your hands immediately.

Example of doffing a mask:

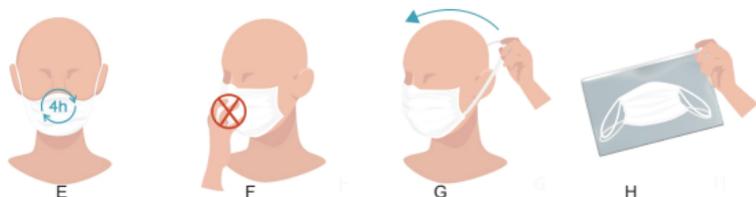

- E) We recommend wearing a mask for no more than 4 hours a day. Change your mask after 4 hours or as soon as the mask is damp. Wash your used mask before reuse.
F) If the mask gets accidentally wet, please exchange it for a dry one as soon as possible. The wet mask must be washed before reuse.
G) The mask must be removed by the elastic loops or ties in the back (do not touch the front of your mask).
H) The mask must be placed inside a closed plastic bag till the time of washing.
I) Wash your hands after doffing a mask.

The masks should not be used in the following situations:

- 1) In any surgical setting or where significant exposure to liquid, bodily or hazardous fluids, may be expected.
- 2) In a clinical setting where the infection risk level through inhalation exposure is high.
- 3) In the presence of a high-intensity heat source or flammable gas

Preservation and washing instructions:

15

Maximum number of washes before the mask loses its protective properties:
15 washes

The masks must be washed using a normal washing cycle at 140°F (domestic wash 6N, following NP EN ISO 6330:2012) with detergent after each use. The user must make sure the mask is completely dry before reuse.



Wash at 140F



Do not use bleach



Do not tumble dry



Iron at 212°F



Do not dry clean

Conformity verified by the report N° 20.09.485.077 ISQ.

www.certifiedfacecovers.com